Hydrofera Blue Classic®

[en] Antibacterial Foam Dressings

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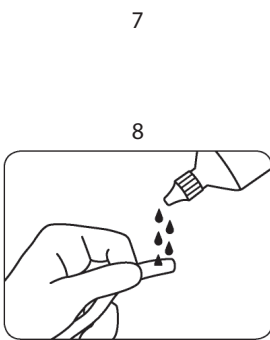
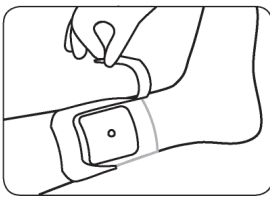
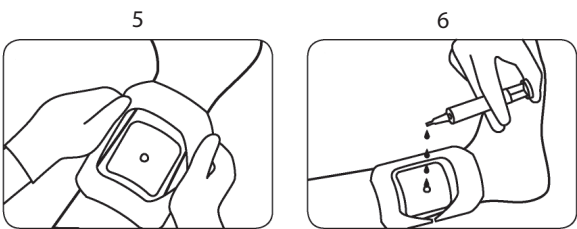
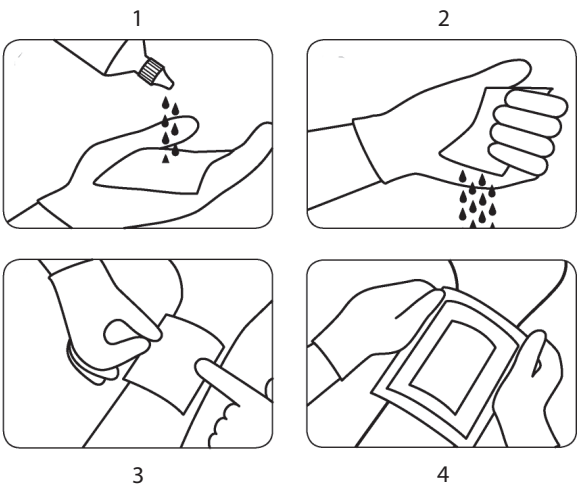
Hydrofera Blue is a registered trademark
of Hydrofera, LLC.

©2024 Hydrofera, LLC. All rights reserved.

08-004-208 Rev. 04

10

Instructions for Use

[en]

Description

Hydrofera Blue Classic antibacterial foam dressing is a sterile absorptive foam dressing made of polyvinyl alcohol (PVA) sponge, methylene blue (less than or equal to 0.00025 g/g), and gentian violet (less than or equal to 0.00025 g/g).

Indications

The antibacterial foam dressings are intended as external dressings for use in local management of wounds such as pressure ulcers, donor sites, venous stasis ulcers, arterial ulcers, diabetic ulcers, abrasions, lacerations, superficial burns, post-surgical incisions, and other external wounds inflicted by trauma.

Contraindications

The dressings are not indicated for third-degree burns.

Application of Hydrofera Blue Classic Dressings Wound Site Preparation

Prior to the initial application of the dressing, the wound should be debrided of any necrotic tissue.

The wound should be cleansed with normal saline and/or an appropriate cleansing solution.

Dressing Application

Hydrofera Blue Classic Standard Dressing

1) Open the dressing package, keeping the dressing on the inside of the package.

2) The dressing may be cut with sterile scissors to fit the wound bed. An appropriate liquid barrier film wipe may be used on the periwound skin.

3) If the wound presents with surrounding erythema or rolled wound edges (epibole), the dressing should be larger than the wound to cover the periwound skin.

4) Moisten the dressing with sterile saline or sterile water (Figure 1) and squeeze out the excess (Figure 2).

5) Apply the dressing to the wound (Figure 3). The dressing may be folded or layered to ensure the desired fit.

Note: Hydrofera Blue Classic dressings with moisture retentive film may not be folded or layered.

6) Secure with a secondary dressing to prevent displacement and to maintain moisture balance (Figure 4).

Examples of secondary dressings (based on exudate level) include transparent film, impregnated gauze, foam dressing, absorbent pad, or gauze.

Hydrofera Blue Classic Moisture Retentive Film Dressings

1) Remove the dressing from the package and hydrate the exposed foam side with approximately 25 - 30 cc of sterile saline or sterile water (Figure 1). Squeeze out excess sterile saline or sterile water (Figure 2).

2) Place the hydrated foam side directly to the wound with the film side on top (Figure 3).

3) Secure with a secondary dressing to prevent displacement and to maintain moisture balance (Figure 4).

Hydrofera Blue Classic Island Dressings

1) Remove the dressing from the package and peel off the dressing backing.

2) Apply the dry dressing to the wound and lightly press adhesive to the skin (Figure 5).

3) Using a syringe, hydrate the dressing with sterile saline or sterile water through the portal opening on top of the dressing (Figure 6).

4) Remove the outer frame backing from the dressing and ensure the adhesive is secured to the skin (Figure 7).

Hydrofera Blue Classic COMFORTCEL™ Interface Dressings

1) Open the dressing package, remove the dressing from the inner bag.

2) The dressing may be cut with sterile scissors to fit the wound bed. An appropriate liquid barrier film wipe may be used on the periwound skin.

3) If the wound presents with surrounding erythema or rolled wound edges (epibole), the dressing should be larger than the wound to cover the periwound skin.

4) Apply the dressing to the wound (Figure 3). The dressing may be folded or layered to ensure the desired fit.

5) Secure with a secondary dressing to prevent displacement and to maintain moisture balance (Figure 4).

Hydrofera Blue Classic Tunneling Dressings

1) Remove the tunneling dressing from the package and moisten the tip with sterile saline or sterile water to aid with ease of insertion (Figure 8). It is important to keep the packing rigid for placement.

2) Gently insert the moistened end of the tunnel dressing (softened) into the wound canal (Figure 9). If necessary, a cotton swab may be used to help guide it into place.

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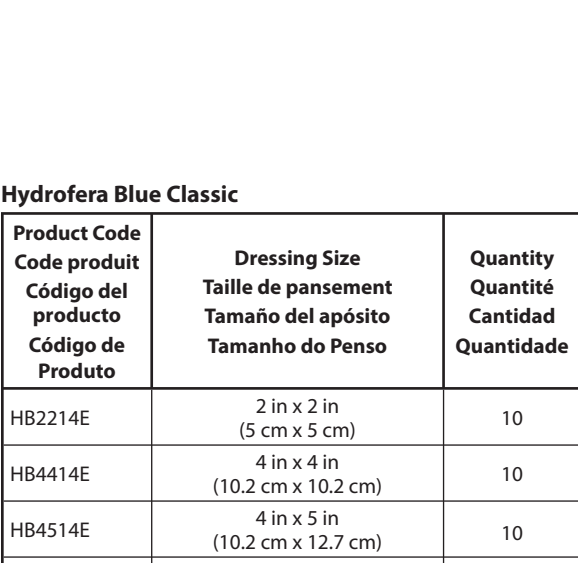
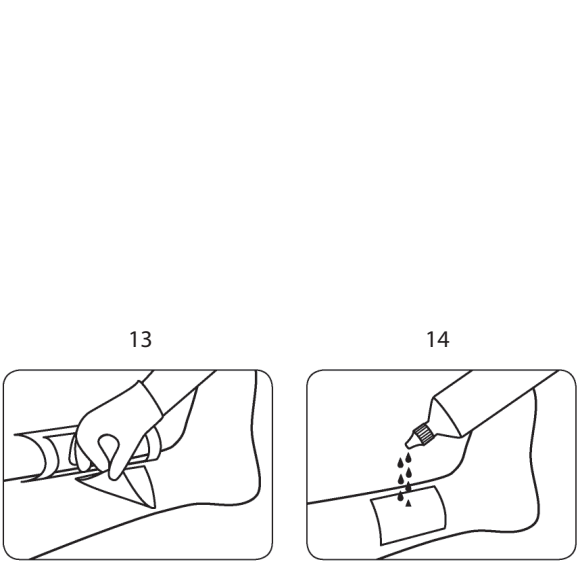
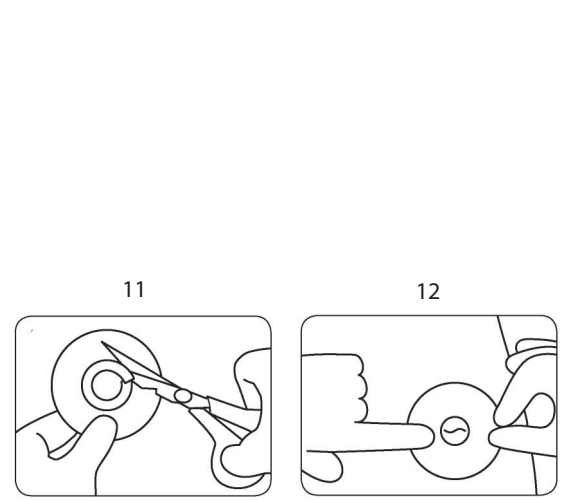
Note: All dimensions of all Hydrofera Blue Classic dressings that are shown represent the product in its hydrated state.

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[en]

3) Leave approximately 1 cm of the tunnel dressing exposed to aid with removal. The dressing can be cut with sterile scissors (Figure 10).

4) After insertion, add additional sterile saline or sterile water to the end of the dressing so the entire tunnel dressing is moist.

5) Secure with a secondary dressing to prevent displacement and to maintain moisture balance.

Hydrofera Blue Classic Ostomy Dressings

1) Trim the inner diameter of the Hydrofera Blue Classic ostomy dressing to fit around the stoma site as needed (Figure 11).

2) Cut the outer diameter of the dressing slightly smaller than the ostomy pouch adhesive area as needed.

3) Place the foam side directly to the wound with the film side on top (Figure 12).

4) The ostomy barrier should be placed over the Hydrofera Blue ostomy dressing and the barrier edges should adhere directly to the patient’s skin.

Dressing Change and Removal

The dressing can be left in place up to 3 days depending on the clinical condition of the wound. Duration of use is determined by the Healthcare Provider and depends on wound type and conditions.

First Dressing Change

The first dressing change should occur at 24 hours.

Examine the area of the dressing in contact with the wound bed (Figure 13):

- If the dressing has turned white or lightened in color, the dressing should be changed again at 24 hours until the dressing retains its color.

- If the dressing has retained its color where it is in contact with the wound, a new dressing can be applied and left in place for up to 72 hours.

Do not allow the dressing to completely dry out.

- The dressing should be rehydrated as needed (Figure 14).

- If the dressing is allowed to dry out, rehydration with sterile saline or sterile water is recommended prior to removal (Figure 14).

- The dressing should be changed if it turns white or upon strikethrough drainage.

Precautions and Observations

- This material is not an implant.

- Infections:** All wounds can be expected to contain some microorganisms; however, if clinical symptoms should develop, appropriate steps as defined by the attending physician should be taken to address the infection. Regular evaluation and cleansing of infected wounds should be a common practice.

- Underlying Causes:** Some wounds, such as leg ulcers and pressure ulcers, can be nonhealing unless steps are taken to correct the underlying pathology. Hydrofera Blue Classic dressings are designed for local wound management and to be part of the overall management program for these types of wounds.

- Drying Out:** Hydrofera Blue Classic dressings are designed to maintain a moist wound environment that will facilitate wound healing. If the dressing is allowed to dry out, rehydration with sterile saline or sterile water is recommended prior to removal.

Note: Hydrofera Blue Classic dressings have two organic pigments bound to the foam to render it antibacterial. The dressing has been tested in vitro and found to be an antibacterial barrier for the following bacteria and yeast: Methicillin Resistant Staphylococcus aureus (MRSA), Staphylococcus aureus, Staphylococcus epidermidis (coagulase neg.), Pseudomonas aeruginosa, Pseudomo-nas florescens, Escherichia coli, Enterococcus faecalis, Vancomycin Resistant Enterococcus (VRE), Streptococ-cus pyogenes, Klebsiella pneumonia, Proteus mirabilis, Proteus vulgaris, Enterobacter aerogenes, Bacillus subtilis, Yersinia enterocolitica, Serratia marcescens, Candida albicans, Candida krusei, and Candida glabrata.

How Supplied

Each dressing is supplied sterile in a pouch. Not made with natural rubber latex.

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