INSTRUCTIONS	Hydrofera <b>BLUE</b> <sup>®</sup>
MODE D'EMPLOI	<span></span>
INSTRUCCIONES	<span></span>



## Hydrofera Blue READY<sup>®</sup>

[en] Antibacterial Foam Dressing

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## Rx Only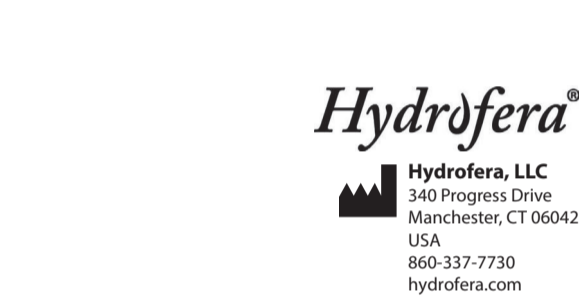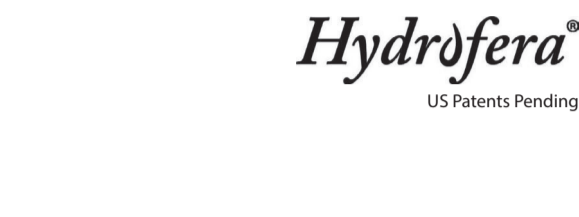

Hydrofera, Hydrofera Blue and Hydrofera Blue Ready are trademarks of Hydrofera, LLC. ©2024 Hydrofera, LLC. All rights reserved.

08-004-209 Rev. 04

[en]

### Description

Hydrofera Blue Ready<sup>®</sup> antibacterial dressing is a sterile absorptive foam dressing made of polyurethane (PU) foam, methylene blue (less than or equal to 0.00035 g/g), and gentian violet (less than or equal to 0.00035 g/g).

### Indications

Hydrofera Blue Ready antibacterial foam dressings are intended as external dressings for use in local management of wounds such as pressure ulcers, donor sites, venous stasis ulcers, arterial ulcers, diabetic ulcers, abrasions, lacerations, superficial burns, post-surgical incisions, and other external wounds inflicted by trauma. Hydrofera Blue Ready antibacterial dressings are indicated for moderate to heavily exuding wounds.

### Contraindications

Hydrofera Blue Ready antibacterial foam dressings are not indicated for third-degree burns and should not be used on individuals who are sensitive to or who have had an allergic reaction to the dressing or one of its components.

### Application of Hydrofera Blue Ready Antibacterial Foam Dressing Wound Site Preparation

- Cleanse the wound in accordance with established procedures.
- Carefully dry the periwound tissue.

### Dressing Application

### Hydrofera Blue Ready Standard Dressing

- Select the appropriate dressing size to ensure the dressing will cover the entire wound.
- The dressing may be cut in order to optimize placement over wounds on heels, elbows etc.
- Apply the dressing with the foam side towards the wound and shiny film side up.
- Secure the dressing using a bandage or a tape over the film surface.
- The dressing can be applied under compression and total contact casting.

### Hydrofera Blue READY – TSD (Tube Site Dressing) Dressing Application

- Fit the Tube Site Dressing around the stoma, tracheostomy tube, or tube site.
- Apply the dressing with the foam side towards the tube site and with the shiny, film side up.
- Additional fixation such as Hypafix<sup>®</sup> Non-Woven Adhesive Dressing Sheet may be used to secure the dressing and prevent displacement.
- During the early stages of treatment, the tracheostomy site should be inspected frequently.

### Dressing Change and Removal

Hydrofera Blue Ready antibacterial foam dressings can be left in place up to 7 days, depending on the exudate and clinical conditions of the wound. The dressing should be changed if it turns white or upon strikethrough drainage.

If the dressing has retained its color where it is in contact with the wound, the dressing may be left in place for up to seven days.

### Precautions and Observations

- This material is not an implant.
- Do not reuse the dressing.
- Infections:** All wounds can be expected to contain some microorganisms; however, if clinical symptoms should develop, appropriate steps as defined by the attending physician should be taken to address that infection. Regular evaluation and cleansing of infected wounds should be a common practice.
- Underlying Causes:** Some wounds, such as leg ulcers and pressure ulcers, can be nonhealing unless steps are taken to correct the underlying pathology. Hydrofera Blue Ready foam dressings are designed for local wound management and to be part of the overall management program for these types of wounds.

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[en]

**Note:** Hydrofera Blue Ready foam dressings have two organic pigments bound to the foam to render it antibacterial. The dressing has been tested in vitro and found to be an antibacterial barrier for the following bacteria and yeast: Methicillin Resistant Staphylococcus aureus (MRSA), Staphylococcus aureus, Staphylococcus epidermidis (coagulase neg.), Pseudomonas aeruginosa, Pseudomonas fluorescens, Escherichia coli, Enterococcus faecalis, Vancomycin Resistant Enterococcus (VRE), Streptococcus pyogenes, Klebsiella pneumoniae, Proteus mirabilis, Proteus vulgaris, Enterobacter aerogenes, Bacillus subtilis, Yersinia enterocolitica, Serratia marcescens, Streptococcus pneumoniae, Candida albicans, Candida krusei, and Candida glabrata. All safety and effectiveness testing is on file at Hydrofera LLC.

### How Supplied

Each dressing is supplied sterile in a pouch. Not made with natural rubber latex.

Product Code	Dressing Size	Quantity
HBRS2520E	2.5 in x 2.5 in (6.4 cm x 6.4 cm)	10
HBRS4520E	4 in x 5 in (10.2 cm x 12.7 cm)	10
HBRS8820E	8 in x 8 in (20.3 cm x 20.3 cm)	10
HBRS3535E	3.5 in x 3.5 in (8.9 cm x 8.9 cm)	30

### Packaging Symbols

Refer to the symbol glossary at:  
[hydrofera.com/products-symbols](http://hydrofera.com/products-symbols)

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